

BIOCARD PRO COVID-19 **RAPID ANTIGEN**

Immediate on-site Covid 19 Antigen Testing

- Based on Colloidal Gold Chromatographic Immunoassay.
- Rapid Qualitative detection of Covid 19 Antigen.
- Easy procedure & Fast Test Time (7~10 minutes).
- Sample Type: Human Nasopharyngeal Swab.
- All Material provided in Kits to perform assay / No Equipment required.
- Room Temperature Storage (4-30°C).




Approved by



icmr
INDIAN COUNCIL OF
MEDICAL RESEARCH
Serving the nation since 1911

BIOCARD Pro COVID-19 Rapid Antigen

BIOCARD Pro COVID-19 Rapid Antigen kit is a chromatographic immunoassay for the qualitative detection of specific antigens to COVID-19 present in human nasopharyngeal swab

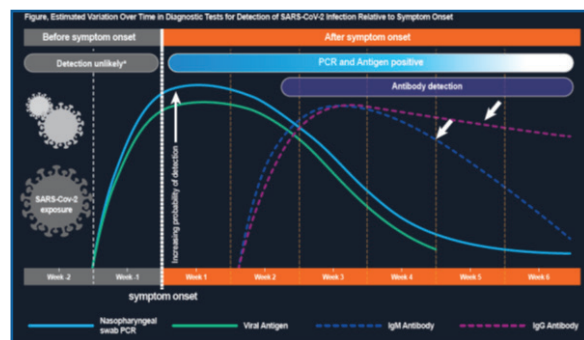

Assay Procedure

Step 01

Add 15 to 20 drops of Sample Extraction Buffer

Step 02

Insert the patient swab sample into the extraction solution tube. Then, swirl swab at least 10 times

Step 03

Remove the swab while squeezing the liquid from the swab by pressing tube side

Step 04

Cover the tube with a filter cap and tighten the lid

Step 05

Incubate the tube for 1-2 minutes

Step 06

Add 2 drops of the extraction solution with specimen into the sample well

Read results after 7~10 minutes

Interpretation of Results

Negative

Positive

Invalid

*Please read result within 15 Min

Specifications

Method	: Immuno Chromatography	Assay Time	: 7 ~ 10 Minutes
Certification	: ICMR Approved	Shelf Life	: 24 Months
Sample Type	: Nasopharyngeal Swab	Storage Temperature	: 4-30°C

Ordering Information

Product Code	Product Description	Pack Size
30201313	BIOCARD Pro COVID-19 Rapid Antigen Test	25 Tests
30201414	BIOCARD Pro COVID-19 Rapid Antigen Test	100 Tests

*Please note: Product specifications are subject to change without prior notice owing to product modifications, improvements / up-gradation.

corporate@trivitrion.com | www.trivitrion.com

Customer Care Helpline: +91 98400 80008

